技术规格/Technical Data Sheet

E-40 HF DF-151SB-T L

MONOBLOCK E-40 HF DF-151SB-T L

编码 / CODE: **30.33.002.007**

S/N: **HF**

IEC601-1: **Class I Type B**

TOSHIBA DF-151SBR X-RAY TUBE

编码 / CODE: **61.18.013.005**

S/N:

小焦点 / SMALL FILAMENT: **0.5 mm**

大焦点 / LARGE FILAMENT: **1.5 mm**

灯丝电源 / FILAMENT POWER SUPPLY: **400 Hz**

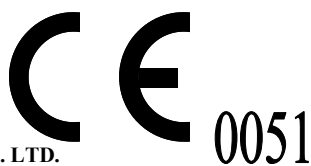
注意/ CAUTION

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WHEN NOT USED, AFTER 3 MONTH PERIOD, PROCEED TO INSERT RESTORATION FOLLOWING THIS CHART:

- 1- FLUORO: FROM 40 kV to 110kV(step 10kV), 1 mA, 5 SEC ON + 5 SEC OFF; PAUSE 5 MIN
 - 2- 90 kV, 5 mA, 15 sec. ON + 5 min. OFF; 110 kV, 5 mA, 15 SEC ON + 5 min OFF
- IF INSTABILITIES OCCUR, PLEASE STOP THE PROCESS, WAIT 5 MINUTES AND RESUME THE PROCEDURE FROM THE BEGINNING.

IMD CHINA CO., LTD.

1-2/F, BUILDING 12, No. 28 YUHUA RD. AREA B, TIANZHU AIRPORT INDUSTRIAL ZONE, BEIJING, CHINA

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X射线管组件性能 / PERFORMANCE OF MONOBLOCK

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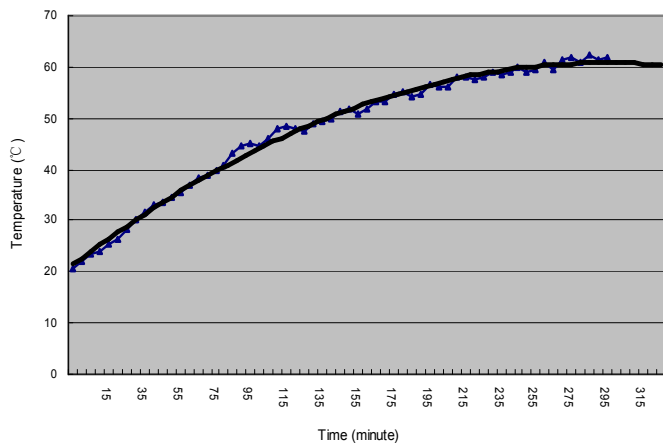
LESS THAN 0.3 mGy IN ONE HOUR (110 kV, 3mA) IN FLUOROSCOPY MODE

ACCORDING TO IEC 601-1-3 APPLICABLE STANDARD (PAR. 12.4)

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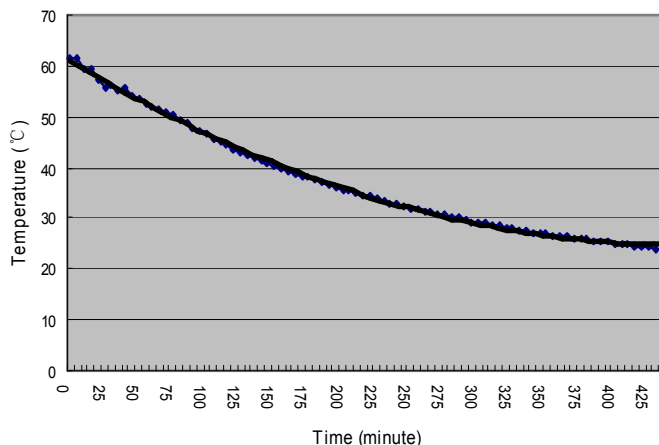
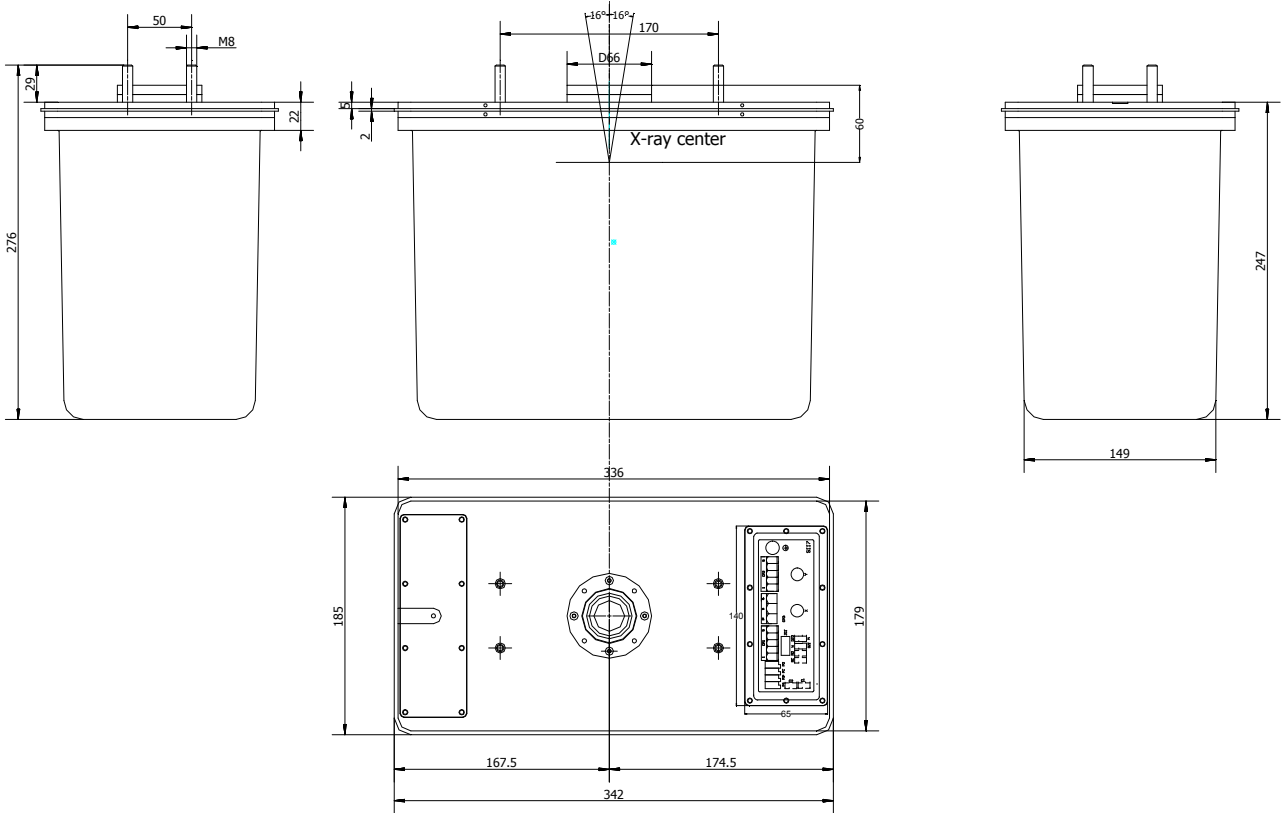
FOR INFORMATION CF. TUBE TECHNICAL DATA SHEET

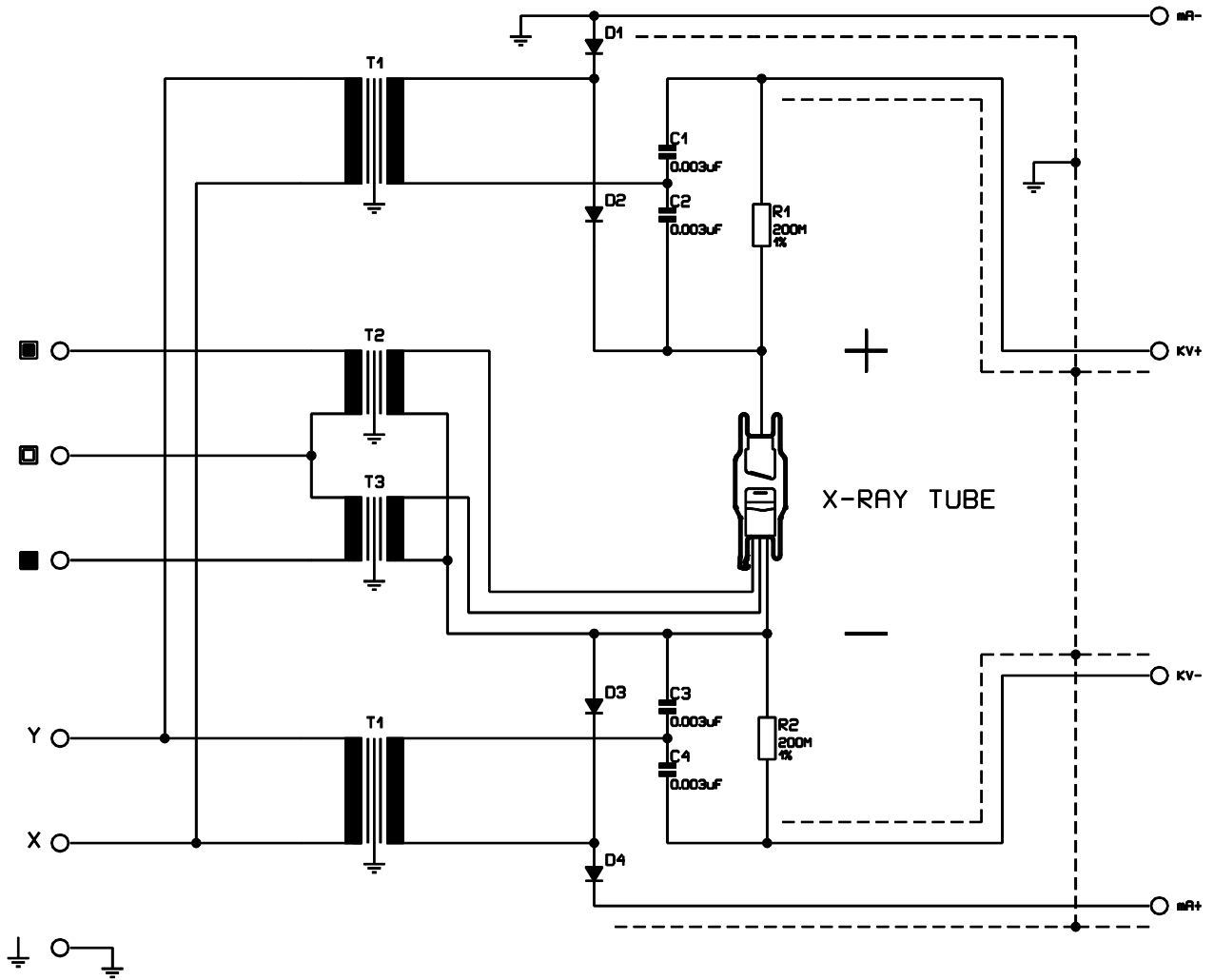
IMD CHINA CO., LTD. • 1-2/F, BUILDING 12, NO. 28 YUHUA RD. AREA B, TIANZHU AIRPORT INDUSTRIAL ZONE, BEIJING, CHINA

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
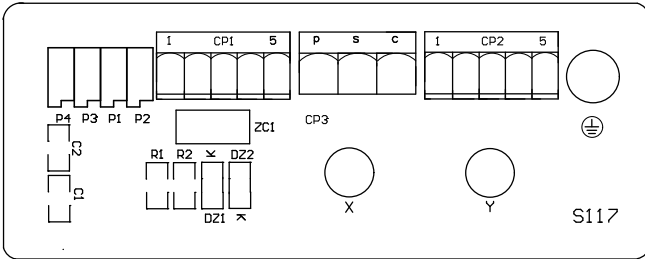
Optionals: 2 disks of aluminium (thickness 1mm), 41/20 o 41/25 lead washer (thickness 2 mm) and seeger.

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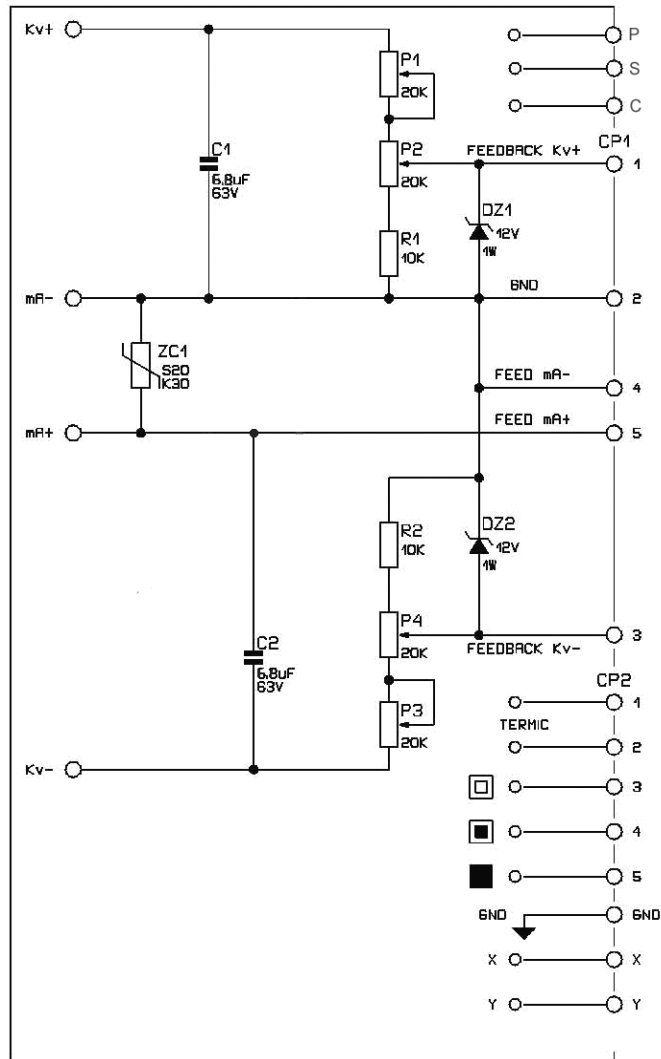E-40 HF DF-151SB-T L

Hazards

PROPER USE AND SAFE OPERATING PRACTICES WITH RESPECT TO X-RAY MONOBLOCK GENERATORS ARE RESPONSIBILITY OF EQUIPMENT MANUFACTURERS AND USERS OF SUCH DEVICES. THE MANUFACTURER PROVIDES INFORMATION ON ITS PRODUCTS AND ASSOCIATED HAZARDS, BUT IT ASSUMES NO RESPONSIBILITY FOR AFTER-SALE OPERATING AND SAFETY PRACTICES. LIMITED LIFE, RANDOM FAILURE AND PERFORMANCE REDUCTION ARE INHERENT CHARACTERISTICS OF X-RAY MONOBLOCK GENERATORS. APPROPRIATE SAFE OPERATING PRACTICES SHOULD BE EMPLOYED.

ALL PERSON WHO WORK WITH MONOBLOCK OR EQUIPMENT WHICH UTILIZES SUCH DEVICES MUST TAKE PRECAUTIONS TO PROTECT THEMSELVES AGAINST RADIATION EXPOSURE AND POSSIBLE SERIOUS BODILY INJURY.

DO NOT OPERATE THIS MONOBLOCK EXCEPT IN ACCORDANCE WITH INFORMATION INCLUDED IN THIS TECHNICAL DATA SHEET, THESE PRECAUTIONS, AND ANY ADDITIONAL INFORMATION PROVIDED BY THE MONOBLOCK MANUFACTURER AND/OR COMPETENT SAFETY AUTHORITIES.

X-RAY MONOBLOCK GENERATOR HAZARDS

HIGH VOLTAGE

All X-Ray Monoblock Generators operate at voltages high enough to kill through electrical shock. Equipment designed should prevent personnel contact with high voltages. When direct access to the Monoblock is required, the primary circuits should be disabled and capacitors discharged.

X-RAY RADIATION

This Device produces X-Ray radiation that varies in quantity and energy level, depending upon the voltage and current used when energizing it. Refer to the operating instructions for the equipment. Operating personnel must be protected by appropriate shielding. X-Ray Caution signs or labels must be permanently attached to equipment directing operating personnel never to operate this device without X-Ray shielding. Shielding should be in accordance with applicable Norms. If there is any doubt as to the adequacy of shielding, an expert in this field should be contacted to perform an X-Ray survey.

Storage and conservation

To guarantee the maintenance of integrity of the product is advised to use or to conserve the system in environments that respect the following climatic factors us.

Humidity				Temperature				Atmospheric pressure			
Use		Storage		Use		Storage		Use		Storage	
min.	max.	min.	max.	min.	max.	min.	max.	min.	max.	min.	max.
30%	75%	10%	90%	10 °C	40 °C	-20 °C	70 °C	500hPa	1600hPa	500hPa	1600hPa

PRECAUTION SHOULD BE TAKEN IN THE DISPOSAL OF X-RAY MONOBLOCK GENERATORS CONTAINING DIELECTRICAL OIL AND/OR LEAD IN ORDER TO COMPLY WITH ALL APPLICABLE EUROPEAN LAWS AND REGULATIONS. IT IS RECOMMENDED THAT FAILED DEVICES BE RETURNED TO THE MANUFACTURER OR AN APPROPRIATE FACILITY TO ENSURE PROPER HANDLING.

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EU DECLARATION OF CONFORMITY (2011/65/EU)

According to Directive 2011/65/EU (RoHS-2), on restriction of the use of certain hazardous substances, the substances listed in Annex II., and the maximum concentration values tolerated by weight in homogeneous materials that are limited as follow:

Lead (0,1 %)

Mercury (0,1 %)

Cadmium (0,01 %)

Hexavalent chromium (0,1 %)

Polybrominated biphenyls (PBB) (0,1 %)

Polybrominated diphenyl ethers (PBDE) (0,1 %).

The company

IMD CHINA Co., Ltd.

DECLARE

under its sole responsibility that the supplied products are in compliance with the European Directive 2011/65/EU (RoHS-2) on the restriction of the use of certain hazardous substances in electrical and electronic equipment.

This statement is based on the control of the internal processes and the evaluation of the reliability of suppliers and materials used for the construction, following the guidelines of the harmonized standard:

EN50581:2012 "Technical documentation for the assessment of electrical and electronic products with respect to the restriction of hazardous substances".

For the realization of the product were applied the following exemptions:

Annex III "Applications exempted from the restriction in Article 4 (1)"

Exemption n° 6a: "Lead as an alloying element in steel for machining purposes and in galvanised steel containing up to 0,35 % lead by weight"

n° 6b: "Lead as an alloying element in aluminium containing up to 0,4 % lead by weight"

n° 6c: "Copper alloy containing up to 4 % lead by weight"

Annex IV "Applications exempted from the restriction in Article 4 (1) specific to medical devices and monitoring and control instruments"

Exemption n° 5: "Lead in shielding for ionizing radiation"

n° 13: "Lead in counterweights"

General Manager